

# SOLACE BRACING Palm Protector

## INDICATIONS FOR USE

- » Protects the palm from finger curling and hand contractures caused by neurological conditions

## MEASUREMENT

- » Palm circumference

## FITTING INSTRUCTIONS

- » Undo all hook and loop closures
- » Place the hand on top of the palm protector with the thumb through the thumb hole
- » The palm protector is sewn at the base so this section should be nearest to the wrist (please take care not to fit this upside down)
- » Secure hook & loop straps around the back of the hand
- » Make sure the straps do not dig into the skin
- » You may need help to fasten the strap depending on your ability or condition

## WASHING INSTRUCTIONS

- » Hand wash only
- » Air dry only

For more information  
visit our site



## Product Code Guide

Size	Code
Standard Left	SOL.SPP/L
Standard Right	SOL.SPP/R
X-Small Left	SOL.SPP/L/XS
X-Small Right	SOL.SPP/R/XS
Standard Pair	SOL.SSP.X2
X-Small Pair	SOL.SPP.X2/XS



### Solace Bracing C/O Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: [info@solacebracing.com](mailto:info@solacebracing.com) W: [solacebracing.com](http://solacebracing.com) T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd  
Sutton House, Bishopstown Road  
Cork, T12 Y9TC, Ireland



Any product supplied by Solace Bracing is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your doctor or clinician for further advice.